

Mobile App Android Pairing Instructions

Step 1


Make Sure **Bluetooth** is Turned On and **Wi-Fi** is Connected

Step 2




Turn On Your **Zerigo Device**
with a **Single-Click**

Step 3

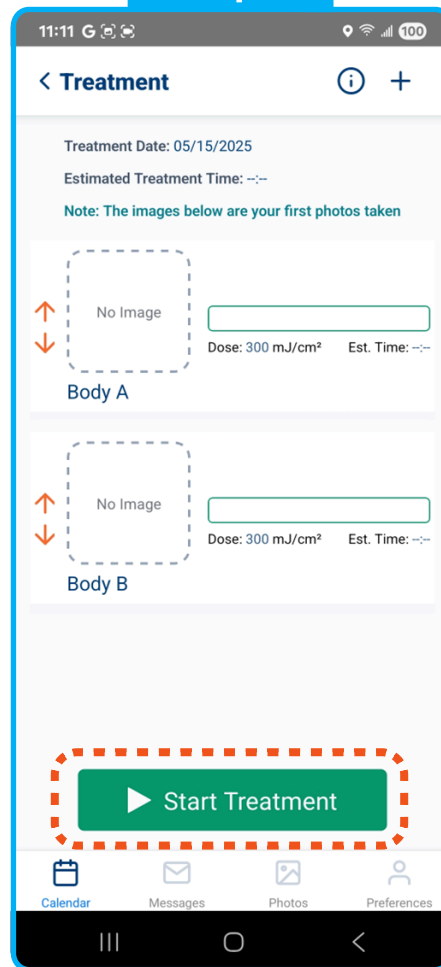

On the Treatment Screen in
the Calendar Tab of the App,
Tap **Start Treatment**

Step 4

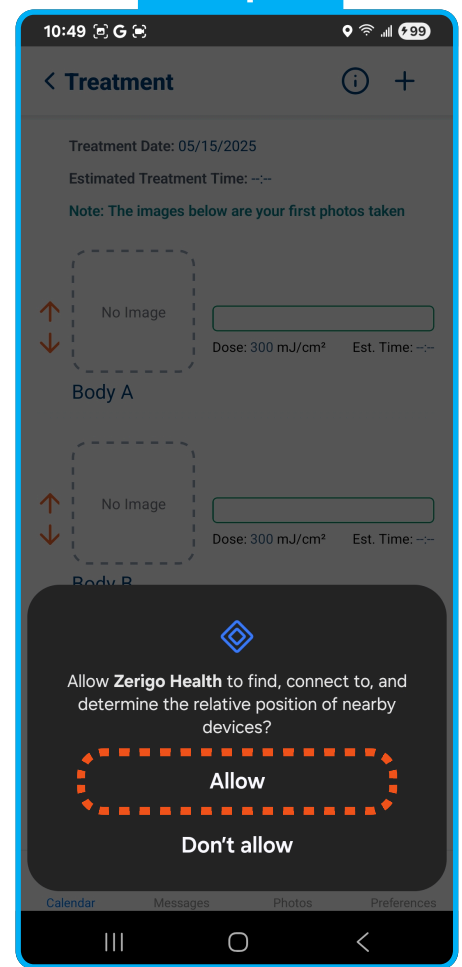

Tap **Allow**
in the App


Mobile App Android Pairing Instructions

Step 5



Type the Numbers Shown on Your Zerigo Device Screen into Your Mobile App and then Tap **Pair** and Wait.

Step 6

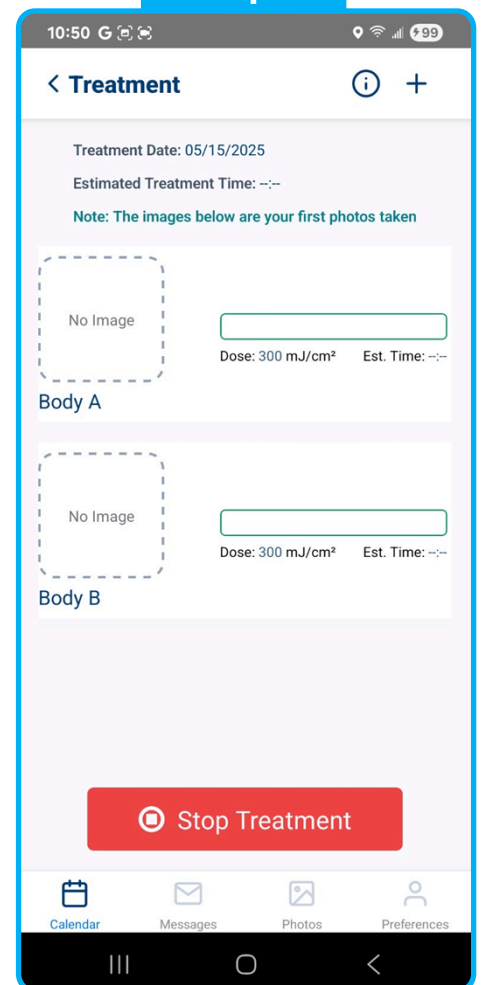

When the **Treatment** Screen in the App Shows **Stop Treatment**, Your Device is Paired!