


Mobile App iPhone Pairing Instructions

Step 1

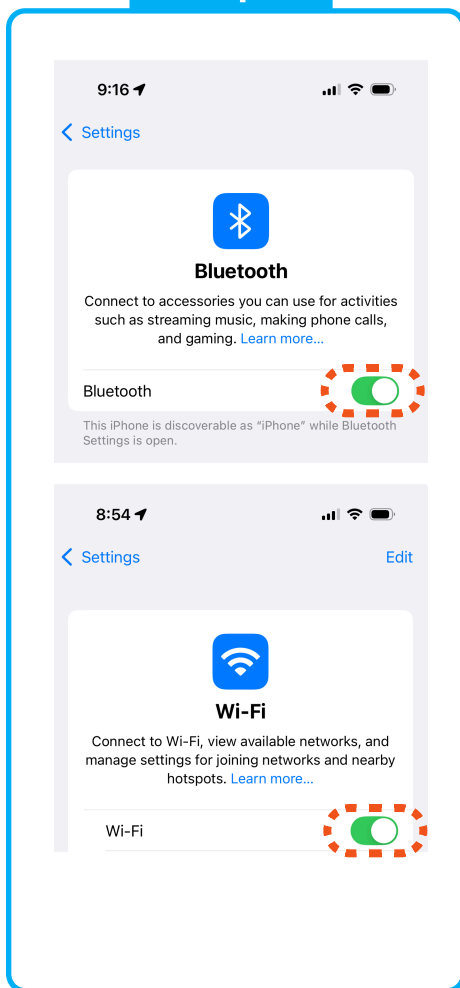


Make Sure **Bluetooth** is
Turned On and **Wi-Fi**
is Connected

Step 2




Turn On Your **Zerigo Device**
with a **Single-Click**

Step 3

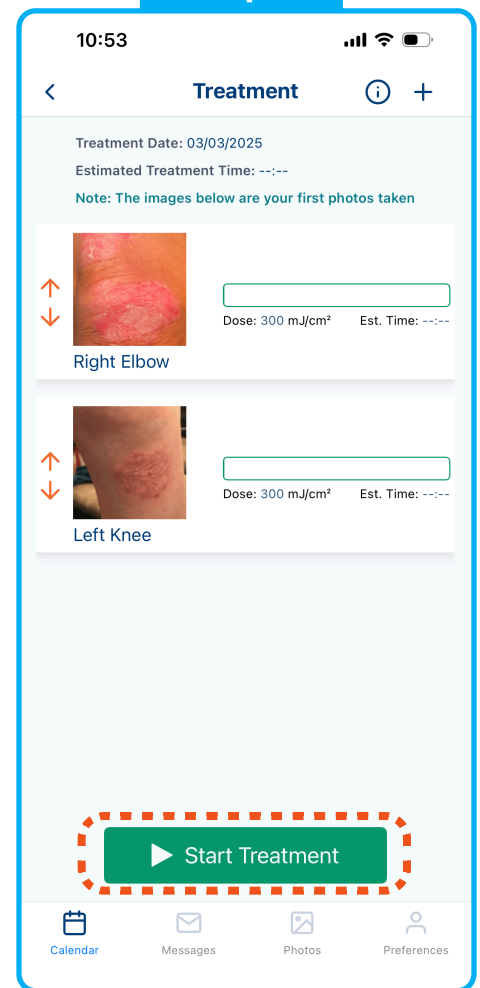


On the Treatment Screen in
the Calendar Tab of the App,
Tap **Start Treatment**

Mobile App iPhone Pairing Instructions

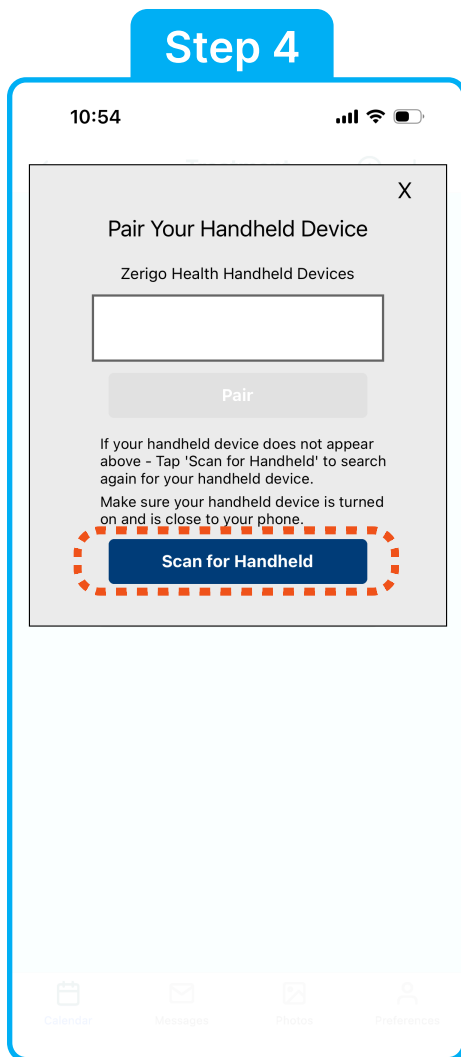

Tap **Scan for Handheld** in the App

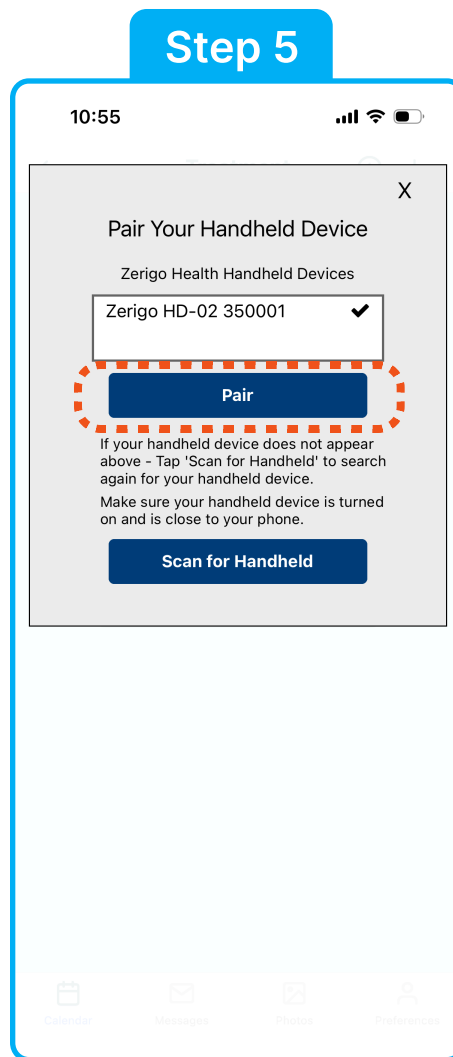

Your Device's Serial Number Will Appear. Tap **Pair**.




Tap **Start Treatment** Again In the App

TIP: If your device's serial number does not appear on the screen, go to Apps section of your iPhone Settings and confirm that the Zerigo Health App is allowed access to Bluetooth.

Mobile App iPhone Pairing Instructions

Step 7

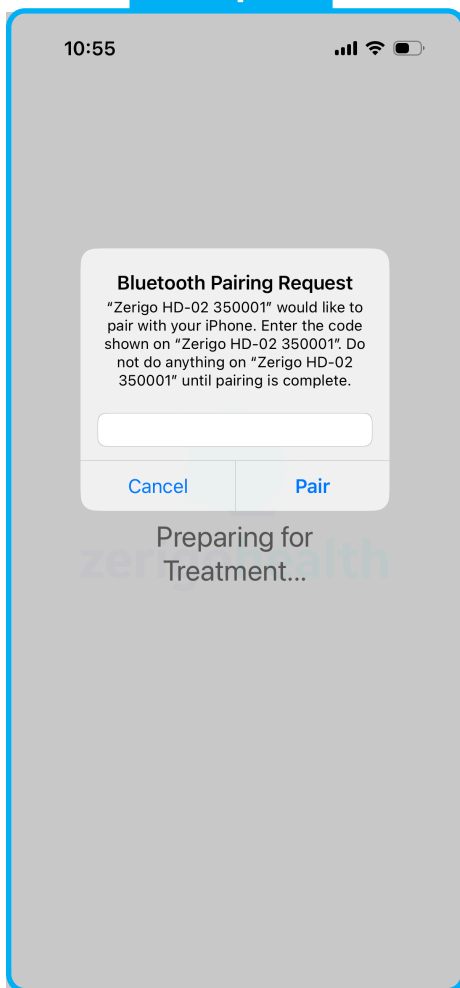

Step 8



A **Bluetooth Pairing Request** Will Appear in the App

Type the Numbers Shown on Your Zerigo Device Screen into Your Mobile App and then Tap **Pair** and Wait. When the **Treatment** Screen in the App Reappears, Your Device is Paired!